

Curera® Positioning Roll

All-round pillow for relief & positioning



 **curera**[™]
positionering

 **care of sweden**

SUPPORTING LIFE

Curera® Positioning Roll

If a person makes too few spontaneous movements, there is an increased risk of pressure ulcers. Regular changes of position reduce the risk of pressure ulcers. Position as required in both reclining and seated position.

Curera® Positioning is a range of stable pillows, rolls, and wedges that meet most needs for support and relief. They are also a great help when changing dressings and for patient care.

Our hygiene covers and cotton covers are also available as accessories to the pillows and rolls.

The hygiene cover does not allow liquid or air to penetrate, but is vapour permeable (vapour is wicked away). Make sure that the patient is positioned correctly so that airways are unobstructed.

Working with Curera® Positioning is simple. Contact Care of Sweden for training and further information.

POSITIONING EXAMPLE




All-round support in seated or reclining position. Used in connection with reduced body balance and to enhance body awareness.



All-round support in seated or reclining position. Used in connection with reduced body balance and to enhance body awareness.



For rest and relief, and for reduced body awareness.

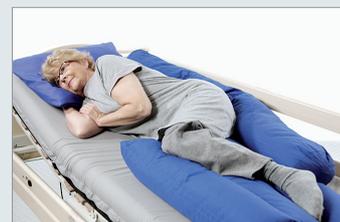

As support in lateral position.



To support shoulder, spine, hips, buttocks and heels.



As support when in seated position. Provides support for the arms.



In connection with swollen arms and hands.



To relieve the knees and support the feet.

CLEANING

The hygiene cover can be wiped clean with detergent and/or disinfectant. Machine wash at max. 95 °C.

Inner pillow: Machine wash at max. 95 °C. Can be tumble-dried or dried in an airing cupboard; avoid over-drying. For faster drying, run the spin cycle three or four times.

Cotton cover: Machine wash at max. 70 °C.

FACTS

- **Filling:** 50 % HR foam/ 50 % polyester fibre
- **Warranty:** 1 year
- **Sizes:** 20 x 225 cm, 20 x 125 cm, 20x80 cm & 15 x 60 cm

Accessories:

Mattress covers: Polyurethane/polyester

Cotton cover



Always read the instruction for use prior to usage of the product.

Care of Sweden and Curera are registered trademarks of Care of Sweden AB. © Care of Sweden AB, 2019. All rights reserved. Reproduction prohibited. Last updated: 2021-05-31